# GEBRAUCHSANWEISUNG INSTRUCTIONS FOR USE INSTRUCTIONS D'UTILISATION

Mobility Plus Package

**e-fix® E35/E36**

# INHALT

## 1. ALLGEMEINES

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# CONTENTS

## 1. GENERAL

Welcome to the Mobility Plus Package	19
Brief overview of the e-fix E35/E36 Mobility App	20
Functions of the Mobility Plus Package	21
Requirements	22
Safety information	23

## 2. CONNECTING TO THE E-FIX

Initial installation	24
Activating the Mobility Plus Package	25

## 3. ALL FUNCTIONS IN DETAIL

26

## 4. NOTES/ERRORS

29

# 1. GENERAL

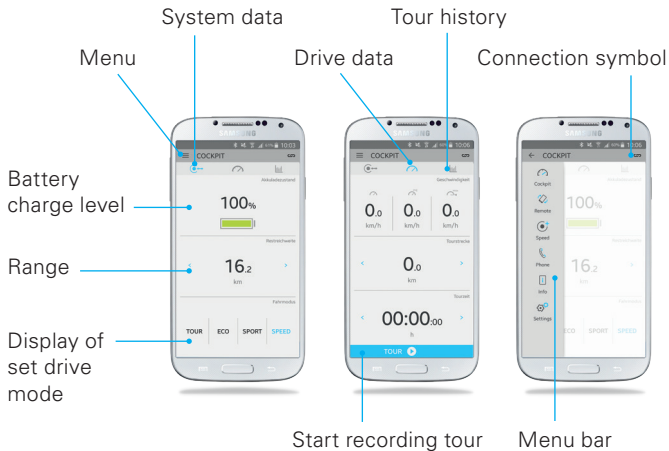
## **Welcome to the Mobility Plus Package**

This package contains a card with a licence code, which can be used to add smart and useful functions to your smartphone via the Alber e-fix E35/E36 Mobility App. Simply scratch to reveal the code, enter this in the app and off you go. We hope you enjoy the new options available to you.

**Note:** Keep your licence code card in a safe place and do not give it to anyone else. Should you lose the licence code, please contact the Alber service centre.

# 1. GENERAL

## Brief overview of the e-fix E35/E36 Mobility App

## Functions of the Mobility Plus Package

### Speed

Increase maximum speed from 6 km/h to 8 km/h

### Remote

Smartphone-based remote control of unoccupied wheelchair

### Phone

Display of a brief summary of incoming calls and texts on the control unit\*

\* Only possible with Android Smartphones.

# 1. GENERAL

## Requirements

The Mobility Plus Package can only be used to add the Mobility App to a smartphone which has Internet access and is Bluetooth® enabled. Another prerequisite is an e-fix E35 / E36 with integrated Bluetooth®, recognizable by the Bluetooth® symbol in the upper left corner of the e-fix display. The Bluetooth® chip has been installed in the e-fix operator panel since September 2017. An e-fix E35 / E36 manufactured before September 2017, can be retrofitted through your specialist medical dealer. This results in additional costs.

Before installing the e-fix Mobility App, make sure it is compatible with the operating system version used on your smartphone (Android or iOS). You can find out which version is required for smooth operation of the Mobility App in the Google Play Store or the Apple App Store.

## **Safety information**

Please read the user manual of your e-fix E35/E36 before using the new functions for the first time and pay special attention to the safety and hazard information contained in the user manual. For the use of the Mobility Plus Package, the following information is to be observed in addition to the general safety information:

- Observe the laws and regulations applicable in the country of use. Where German road traffic licensing regulations apply (StVZO), the speed must not exceed 6 km/h, the 8 km/h assistance speed is only permitted in private areas.
- The wheelchair may only be remotely controlled with the remote function when the wheelchair is not occupied.

Should impermissible use occur, this is considered to be misuse. Alber declines all liability for resulting damages.

## 2. CONNECTING TO THE E-FIX

### Initial installation

1. Download the free e-fix E35/E36 Mobility App (Google Play Store or Apple App Store).
2. Activate Bluetooth® on your smartphone.
3. Press the connection symbol  → a Bluetooth® device is automatically looked for.
4. Select e-fix Bluetooth® code (e.g.: e-fix-1986) if several devices are shown, the e-fix code can be seen in the menu under Settings/Bluetooth®.
5. As soon as a connection has been established with the e-fix control unit, the Bluetooth® symbol in the top left of the e-fix display assumes a blue background.

## **Activating the Mobility Plus Package**

1. Scratch card provided to reveal the licence code.
2. Press one of the inactive Mobility Plus Package functions in the Mobility App → the “Mobility Plus Package activation” screen appears automatically
3. Enter licence code (12 digits)
4. Press “Activate” to confirm
5. Create your own password (4 digits). This password is for the “Remote” function.

If the activation was successful, the menu items of the Mobility Plus Packages are activated. Now the new functions can be selected.

### 3. ALL FUNCTIONS IN DETAIL

#### Cockpit

Display of information, such as charge state of battery pack, tour data and the drive mode currently selected.

#### Speed

Increases the assistance speed from 6 km/h to 8 km/h\*. For possible restrictions on increasing speed, see notes/errors. Country-specific road traffic legislation should be observed.

\* Please note any country-specific laws and regulations. Wherever the StVO (German Road Traffic regulation) applies, the speed may not exceed 6 km/h, 8 km/h support speed is only allowed in private areas.

Using the e-fix at 8km/h in the application area of the StVO requires individual approval according to StVZO and insurance according to the German obligatory car insurance law. No speed increase to 8km/h with Opti-Box.

The range may be reduced as a result of driving 8 km/h.

### Remote

The unoccupied wheelchair can either be controlled using integrated inclination sensor (by tilting smartphone slightly) or a virtual joystick. The speed can be set to between 0.5–3.0 km/h. Moving the joystick or pressing any button deactivates the remote function.

### Phone

A brief summary of incoming calls and texts to the smartphone are shown on the e-fix display.\*

\* Only possible with Android Smartphones.

### 3. ALL FUNCTIONS IN DETAIL

#### **Info**

Information about the e-fix, descriptions of errors with proposed solutions and direct contact with Alber and/or the specialist dealer.

#### **Settings**

Your data and the e-fix data should be stored here. In the event of a problem, this can then be easily e-mailed to your specialist dealer or Alber. More adjustment options: Language (only Android) or units of measurement.

## 4. NOTES/ERRORS

### **Bluetooth® symbol display**

Grey Bluetooth® symbol on the e-fix display: Bluetooth® is active.

Blue Bluetooth® symbol: Devices are linked.

Bluetooth® symbol not visible: Check in the menu of the control unit under Settings/Bluetooth® whether the function is deactivated. If no Bluetooth® can be found in the menu under Settings, Bluetooth® is not installed in the control unit.

Red Bluetooth® symbol: Connection/hardware error, the e-fix must be switched off/on to reconnect the devices. If a connection still cannot be established, refer to the following item.

### **Bluetooth® connection cannot be established**

Activate/deactivate Bluetooth® on your smartphone. Restart the app and e-fix and try to establish a connection. If this doesn't solve the problem, reinstall the app on your smartphone. If even this doesn't solve the problem, contact your specialist dealer or our Alber service.

## 4. NOTES/ERRORS

### **Sources of interference for Bluetooth® connection**

If a Bluetooth® connection cannot be established between your smartphone and the e-fix control unit or if the Bluetooth® connection aborts, there may be other Bluetooth® devices or products operating in a frequency range of 2.4 to 5 GHz (microwave transmitters, wireless cameras, baby monitors, WLAN devices).

### **Aborting remote connection**

If the remote control is active, it can be aborted by pressing any key on the control unit or moving the joystick. The e-fix remains in its current position. If required, the control system can be re-engaged.

### **Remote not available**

You cannot start the remote function if you are in the menu of the e-fix control unit or the intuitive attendant control is activated.

### **Combination with Opti-Box/special control**

The speed mode is deactivated if there is an Opti-Box connected to your e-fix. Drive at 8 km/h is therefore not possible.

### **Programming handling à custom**

The speed mode is deactivated if individual programming has been undertaken by the specialist dealer/therapist. Drive at 8 km/h is therefore not possible. This is indicated by the custom drive mode.

### **How do I reactivate the Mobility Plus Package functions after a smartphone change?**

You can reactivate them with your current license code.

The license code can be used for any number of smartphones.

## 4. NOTES/ERRORS

### **The control unit had a fault and had to be replaced by the dealer?**

Here you need a new license code. The Mobility Plus Package functions can only be activated on one control unit at a time with one license code. Please contact your dealer or Alber directly with proof of purchase (invoice).

### **How do I reset the individual password for the „Remote“ function?**

In the menu of the e-fix app under Settings you will find the option „Reset PIN“. To reset, you need your license code, follow the instructions in the app.

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41.0007.4-99.05 Stand 09/2023 © Alber GmbH, Albstadt
**Alber GmbH**