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Rollator vital carbon plus 62

Art.-Nr. 11448 700 carbonschwarz
 Art.-Nr. 11778 705 saphirblau
 Art.-Nr. 11448 715 rubinrot
 HMV-Nr. 10.50.04.1253

Rollator vital carbon plus 54

Art.-Nr. 11448 701 carbonschwarz
 Art.-Nr. 11778 706 saphirblau
 Art.-Nr. 11448 716 rubinrot
 HMV-Nr. 10.50.04.1253

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Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this product is to be used by other persons, they must be fully informed of these safety warnings before use.

Indication/Contraindication

The rollator is a mobility aid for people with mobility restrictions, e.g. people with a disability that makes walking difficult or who cannot walk for long, providing them with the opportunity to go further afield. Staying on the move not only serves to improve circulation, coordination and balance, but it also enhances the user's well-being.

This rollator is designed both for use in enclosed areas and in the open. It is equipped with a removable bag and cane holder. The rollator provides a seat for short rests. It can be folded easily for transportation, thus saving space. The rollator should not be used by persons with perception impairments, serious problems with their balance or an inability to sit. The user must have adequate perception skills and adequate physical ability to be able to steer a rollator.

Safety Indications

- Talk to your doctor or specialist retailer to see whether this Rollator is optimally suitable for you and your needs.
- Check that the Rollator is in perfect condition every time before you use it. Check the following components and make sure:
 - all screws, nuts and other parts are firmly tightened
 - all wheel surfaces are intact
 - the height adjustment screws are firmly tightened
 - the brakes function both as parking brakes and hand brakes
 - there are no visible signs of wear on the frames, welded joints and plastic elements
 - wheels and wheel forks should be free of dirt and turn smoothly.
- Do not use a damaged or defective Rollator!
- Ensure that the Rollator is correctly engaged before every use.
- Always activate the parking brake when you stop or park the Rollator. Always use the hand brakes slowly, evenly and both together.
- The braking effect can be reduced if the tyre surface is worn. Inspect the braking action of the brakes regularly.
- The Rollator should only be used on firm and even surfaces. There is a risk of slipping on dirty, wet, frosty or snow-covered ground! The braking effect is also limited. Please adapt the way you move and walk to these used in moving traffic.
- On smooth surfaces such as tiles, stone flooring, PVC, etc. the wheels can block when braking under certain circumstances. Therefore please check the braking performance of your Rollator carefully on these floors and adjust the way you move and walk accordingly. If you wish to sit on the Rollator you should also make sure that the wheels have a sufficient hold on the surface when the brakes are applied and that the Rollator cannot slide away.
- The Rollator should not be used as a support when getting up from a crouching or sitting position! It should only be used for its intended purpose and should not be used for transporting objects over 5 kg or persons.
- The maximum permissible load of the rollator is 150 kg. This includes the maximum luggage load of 5 kg in the bag.
- The bag included in the delivery is designed for a max. luggage load of 5 kg. The bag must not be overloaded. Only use the bag for transporting objects.
- Ensure that the handles are adjusted to the correct height for the user in order to avoid improper weight load or falls. Always hold both handles; otherwise


you can lose your balance. We recommend you walk between the handles. When walking, take care not to bump your feet against the wheels or the step stool or your legs against the cross brace.

- Make sure that your hands or clothes do not come into contact with the moving parts of the Rollator! Be particularly careful when folding and unfolding the Rollator. This applies in particular to hinges under the seat or in the middle of the cross brace.
- The Rollator should be used on the pavement. The Rollator should not be used in moving traffic. For your personal safety on the road, it is advisable to wear additional reflectors on your clothing in addition to the reflectors already attached to the rollator on all four sides.
- Do not sit on the rollator when travelling on public transport. Park the rollator with the brakes on so that it cannot fall over or roll away. Look for a seat close by.
- If you have to cross (tram) tracks, If possible, drive over them at a right angle and quickly. over them. Make sure that the wheels do not get caught in the rails. get caught in the rails.
- Be careful near children. Children should not play with the Rollator!
- Do not hang any handbags or carrier bags on the handles, the balance of the Rollator can be impaired if you do.
- Do not pull or carry the rollator by the seat, the folding strap or the brake cables.

- Do not carry out any adjustments or modifications, which impair the safety of the design. Parts such as wheels, handles or brake cables may only be replaced by a specialist retailer.
- Only use accessories approved by the manufacturer. Please also read the instructions for use for each accessory.
- In strong sunlight, parts of the rollator, e.g. the parts of the rollator, e.g. handles or seat, may heat up. In these In these cases, skin contact should be avoided to prevent burns. Make sure to cover your rollator cover your rollator or park it in a shady place. place.
- Please prevent skin contact with the Rollator if the Rollator has been in cold temperature for a long period of time in order to prevent freezing.
- Keep the rollator away from naked flames.
- Please prevent running into door frames, furniture, etc. with the Rollator and check the secure position of the Rollator when parked. The frame, connection elements and welding seams may be damaged if the Rollator tips over.
- Please also prevent placing stress on the Rollator when folded together! This could damage the frame, connection elements and welding seams.
- Do not use the Rollator in stairs or escalators.
- Do not stand on the rollator.
- Do not lean on the brake lever.
- If you have any queries, please refer to your dealer.

Remove all packaging material. Check the product has not been damaged in transit before using it for the first time. Pay particular attention to ensuring that the wheels and wheel suspension, screw and plastic connections and brake cables are intact The rollator is delivered pre-assembled. Before using for the first time, check that the brakes work properly; they may need to be readjusted. The hand grips must be adjusted to the appropriate height for the user. Please dispose of the packaging material correctly.

Contents/Description of the parts:

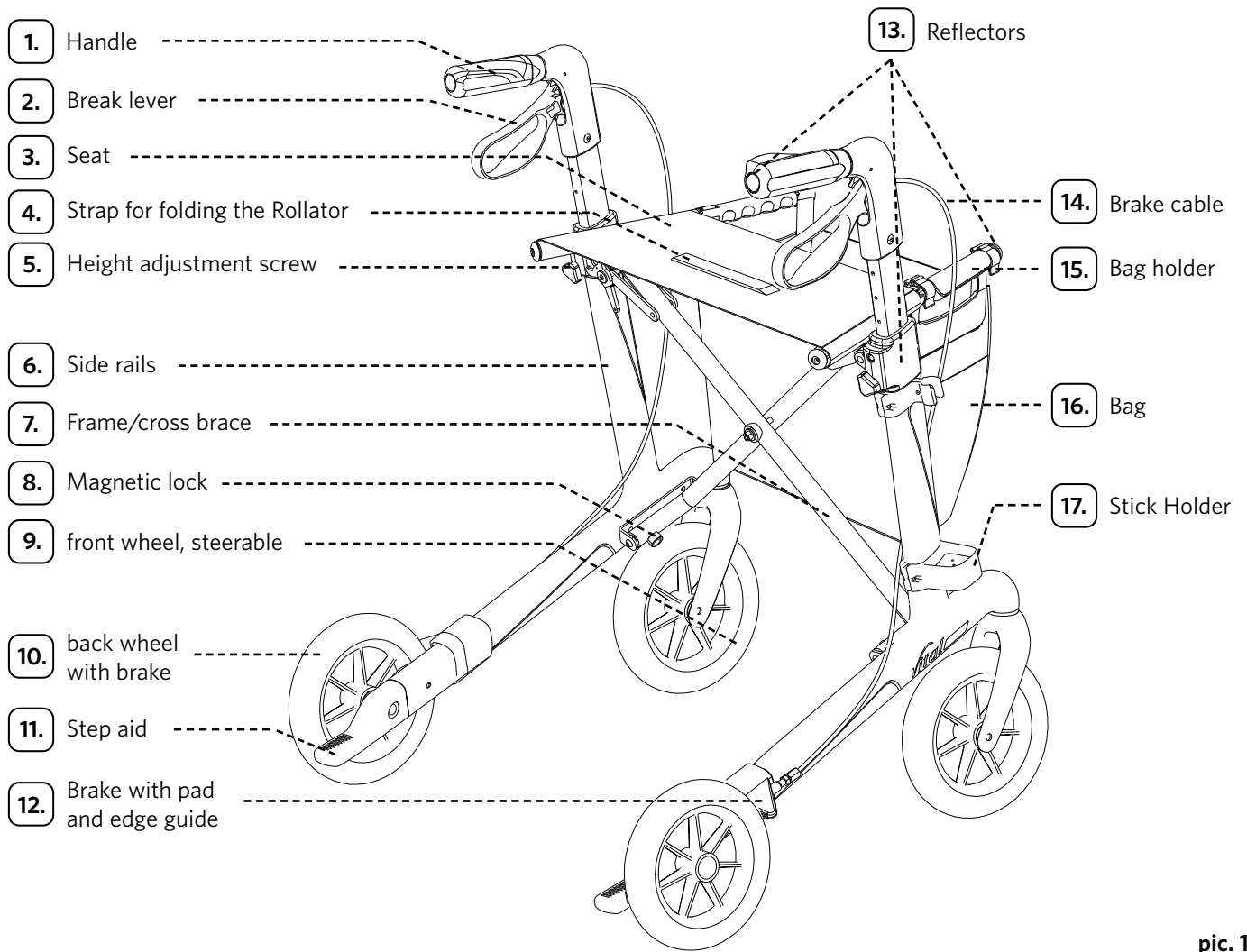pic. 1

Assembly

Opening the Rollator

1. Pull the handles apart (pic. 2).
2. Push the side rails downwards until they engage. The Rollator is now fully opened (pic. 3).
3. Make sure the Rollator is engaged before you use it.

pic. 2

pic. 3

Folding up the Rollator

1. Fold the Rollator up by pulling the strap upwards (pic. 4).
2. Push the handles inwards and the Rollator together (pic. 5). The fully folded Rollator is locked with magnets (pic. 6).
3. When folded, the rollator can easily be carried or pulled by the basket holder (pic.7). Do not carry or pull the rollator by the seat, the strap for folding or the brake cables.

pic. 4

pic. 5

pic. 6


pic. 7

Height adjustment of the handles (pic. 8)

Adjust the height of the handles so that the user can walk upright, without bending over. It is recommended that the user stands upright with arms at their sides between the handles. The grips should be adjusted to the height of their wrists.

1. Loosen the height adjustment screw (pic. 1, No. 5).
2. Move the handle to the desired height.
3. Tighten the height adjustment screw (pic. 1, No. 5).

Adjust both handles in the same height. The printed numbers on the inside of the push handles help you easily adjust your personal grip level again (pic. 8).

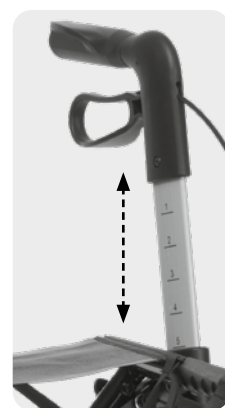pic. 8

Brake (pic. 9)

Check the brake function before use. The built-in brake functions as an operating brake for slowing down when moving. The parking brake can be used when the rollator is stopped.

1. Speed is reduced by pulling the brake lever upwards (operating brake).
2. The parking brake is activated by pressing the brake lever down until it engages.
3. The parking brake is released from its locked position by pulling the brake lever upwards.
4. The rollator must not be pushed while the operating or parking brakes have been applied. If the rollator is pushed while the operating or parking brakes have been applied, the tyres and brakes may be subject to excessive wear and deteriorate.
5. Regularly check that the brakes work satisfactorily. We recommend checking before every use. The rollator must not be used with damaged brakes or damaged wheels.


pic. 9

Brake adjustment (pic. 10)

If the braking force is decreasing due to wear on the wheels, the brake shoe can be adjusted. The brakes must be adjusted so that both wheels brake evenly. The distance between the brake pad and the wheel surface should be about 2–3 mm. After replacing wheels, the brake setting must be checked again and if necessary, readjusted.

1. Loosen the locking nut.
2. Adjust the braking force by turning the brake cable spanner inwards or outwards.
3. Check the braking function by turning the wheels and operating the brake levers. The wheels should turn freely but stop immediately when the brakes are applied. The wheels must not be able to move when the brakes are applied.
4. Tighten the locking nut firmly again.
5. If you are unsure about the correct adjustment of the brakes, please contact your specialist retailer.

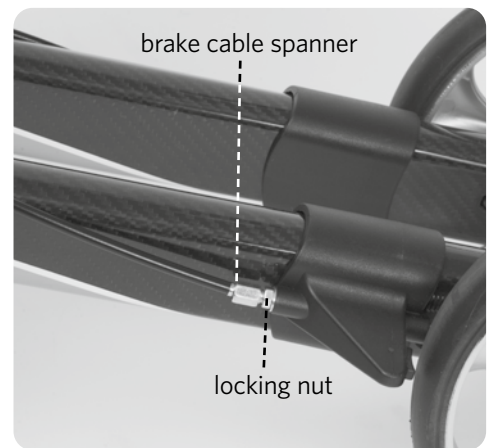pic. 10

Seat

Activate the parking brake before you sit on the seat. Always sit fully on the seat with your back in the direction of travel. Do not lean against the seat! Do not lean over the sides or backwards while sitting on the rollator. That could impair the stability of the Rollator. Do not use the seat on a sloping surface. Ensure that the front wheels are parallel to the direction of walking. The seat has been tested for a user weight of up to 150 kg.

Overcoming obstacles with the help of the step assistant

The step aid is designed to make it easier to negotiate flat steps and kerbs. It is already mounted on both sides of the frame between the rear wheels. Before using the step stool, make sure that you can stand on one leg safely. Practice using it with an assistant if necessary. If you have to negotiate an obstacle, for example a kerb, drive straight towards the obstacle with the walker until the two front wheels are parallel and directly in front of the obstacle (pic. 11). Use both hands to pull the brake levers up so that both rear wheels are braked. Now carefully press down the step stool with your right or left foot and at the same time pull both handles towards you (pic. 12). With the centre of gravity of the walker-rollator above the rear wheels, release the brake levers slowly and simultaneously so that you can roll the walker-rollator on the rear wheels towards the obstacle (pic. 13). Apply the brakes evenly again. Now you can lower the front wheels onto the obstacle with the brakes applied. Release the brakes and push the walker forward until it is completely on the obstacle (pic. 14).

pic. 11

pic. 12

pic. 13

pic. 14

Stick Holder


You can use the stick holder to attach your walking stick simply and securely to the rollator and always have it handy for when you need it. You can either fix the stick holder to the right or left hand frame of your rollator. Unscrew the fixing screws with the tool supplied and place the small basket at the lower end around the vertical frame (pic. 16). Ensure that the basket points outwards. Insert the fixing screw through the opening provided (pic. 17). Tighten the screw so that the stick holder cannot become loose (pic. 18). Proceed with fixing the upper bracket in exactly the same way. It is fixed at the upper end of the vertical rod of the frame.

Keep the tool in a safe place in case you wish to reposition or remove the stick holder.


When you wish to place your stick into the stick holder or remove it, apply the brakes of your rollator firmly so that it cannot roll away. To fix your stick, first place the rubber pad into the lower basket of the stick holder. Then press the shaft of your stick into the upper clip of the stick holder until it is fixed. To remove the stick, pull it firmly out of the clip. An attached stick can project from your rollator. Please be extra cautious so you do not catch the stick on anything or injure other people with it.


pic. 15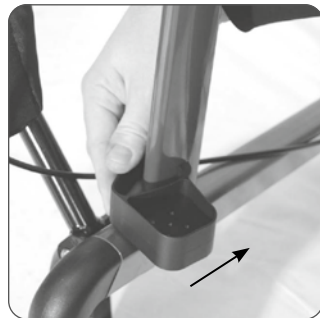

pic. 16

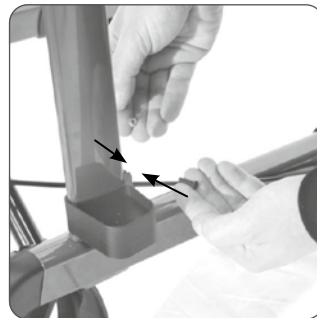pic. 17

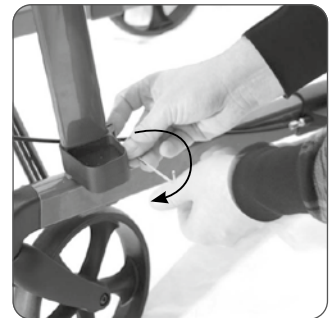pic. 18


General care and cleaning

Rollator

Keep the rollator free of dirt to keep it functional. The rollator can be cleaned with lukewarm water, neutral commercial and household cleaning agents and a soft cloth to remove dirt, then rinsed and dried. No water must get into the pipes. After cleaning or after rain, dry the walker thoroughly with a dry, clean cloth. Also clean the brake pads and springs regularly.


Do not use abrasive, corrosive or bleaching cleaning agents. Do not use metal brushes or other sharp-edged cleaning aids for cleaning. Further cleaning instructions for reuse and before changing patients can be found under the Hygiene Plan.

Do not use the Rollator in temperatures below -35°C or above $+60^{\circ}\text{C}$.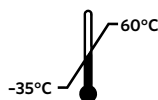

Bag

Small items can be carried conveniently in the bag. The inside pockets are suitable for storing items like keys or your purse or wallet. The bag is designed to carry up to 5 kg. It can be hung up and removed again easily using the integrated handles.

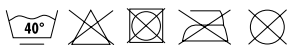pic. 19

Re-use

The product can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again. Please order our maintenance and hygiene guidelines.

Please find below a maintenance table and hygiene plan for the regular care and maintenance of this device:

| Maintenance Table | |
|---|--|
| When | |
| Before every re-use | |
| What | Caution |
| Check general condition Visually inspect all frame parts including welded joints for deformation, damage, wear or corrosion | If you find any defects, replace the affected parts of the device. |
| Check hand grips visually and by attempting to turn them | If you find that the hand grips are damaged or no longer firmly connected to the frame, they must be replaced. |
| Check the tyre tread and ball bearings | If you find that the wheels are worn or cracked, they must be replaced. |
| Check the folding lock | If the folding mechanism no longer works properly, it must be repaired or replaced. |
| Check the connecting elements for tightness | If you find that screws, nuts, rivets, etc. are loose or missing they must be tightened, fitted or replaced. |
| Check the braking system is working satisfactorily Apply the operating brake until it reaches the stopping point. The wheels must not be able to turn now when pushed hard. Apply the parking brake by pressing the brake levers down until it engages. The wheels must not be able to turn now when pushed hard. | If the brakes are defective, the brake slack, brake cables, brake shoes and wheels must be checked and the defective part of the device must be replaced if necessary. |

Send your rollator for regular checks to your dealer. He will be able to detect any wear and tear and replace parts, if necessary. If you find the brake action to be faulty in any way, please contact your dealer immediately.

Hygiene Plan

The rollator must be cleaned and disinfected before each re-use or before each change of patient. The disinfection of the rollator must only be carried out by a specialist.

Cleaning/disinfection

1. Put on protective gloves and, if necessary, protective clothing (disposable apron, mouth and nose protector, etc.) before preparing the rollator.
2. Always clean first, then disinfect.
3. To remove any dirt, the rollator can be cleaned with lukewarm water, neutral, commercial and household cleaning agents and a soft cloth, then rinsed and dried. No water must get into the pipes. Also clean the brake pads and the springs. Do not use abrasive, corrosive or bleaching cleaning agents. Do not use metal brushes or other sharp-edged cleaning aids for cleaning. After using cleaning agents, rinse the rollator thoroughly with clean water. Before disinfecting, there must be no residues of cleaning agents on the surface. After cleaning, dry the rollator with a dry, clean cloth.
4. Wipe or spray the rollator with a disinfectant listed by DGHM/VAH (follow the manufacturer's instructions). The product can be disinfected with a disinfectant containing 70–80 percent ethanol. Disinfectants containing chlorine or phenol must not be used. Wipe disinfection is always preferable to spray disinfection. After the disinfectant has soaked in, dry the walker with a dry, clean cloth.

Storage

Store the Rollator light and dust-protected in a dry place. Do not store near a heat source. Protect the rollator from dirt and contamination after treatment or if it will not be used for a long period of time. ☂️☀️

Disposal

Do not dispose of the rollator in the normal household rubbish. Ask your local recycling authority how to dispose of it properly and in an environmentally friendly way.

Technical Details

Material:

- Frame: Carbon with plastic-segments
- Wheels: Polyurethan (PU)

Rollator vital carbon plus 62

- Dimensions: 61,3 x 66 x 74-101,5 cm (W x L x H)
- Folding measure: 23 x 66 x 79 cm
- Tube diameter of the frame: 23,5 x 49 mm
- Wheels: 20 x 3,6 cm
- Seat level: 62 cm
- Seat: 36,5/45 x 18,5 cm (W x L)
- Distance between handles: 46 cm
- Weight: 6,21 kg + bag (380 g)
- Turning circle: 84 cm
- Recommended body size: 170-210 cm
- Max. loading capacity: 150 kg

Bag

- Material: 100 % Polyester
- Dimensions: 32 x 15 x 28 cm (W x L x H)
- Permissible weight: max. 5 kg

Rollator vital carbon plus 54

- Dimensions: 63,4 x 66 x 66,2-85 cm (W x L x H)
- Folding measure: 24,1 x 66,2 x 74,5 cm
- Tube diameter of the frame: 23,5 x 49 mm
- Wheels: 20 x 3,6 cm
- Seat level: 54 cm
- Seat: 36,5/45 x 18,5 cm (W x L)
- Distance between handles: 45,2 cm
- Weight: 6,06 kg + bag (380 g)
- Turning circle: 85,6 cm
- Recommended body size: 150-180 cm
- Max. loading capacity: 150 kg

Stick Holder

- Material: Plastic
- Measurements: Ø clip: 2,5 cm
- Lower part: 4,4 x 4,4 cm

MD This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking. **CE**

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@russka.de as well as the national competent authority in your country.

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Fabrikant: Ludwig Bertram GmbH
Albo-Healthcare GmbH